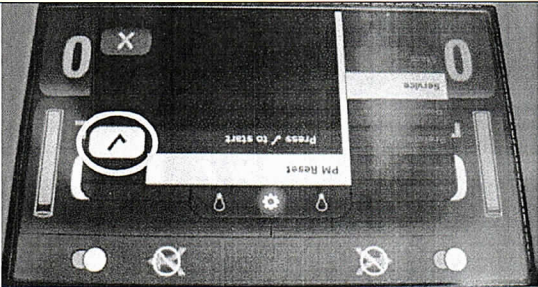
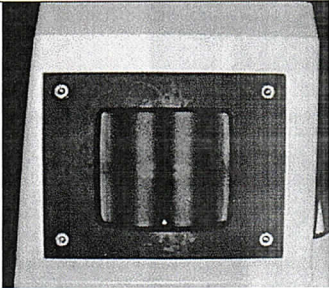
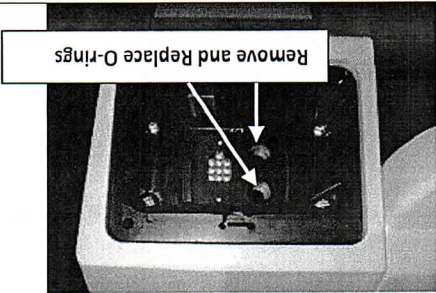
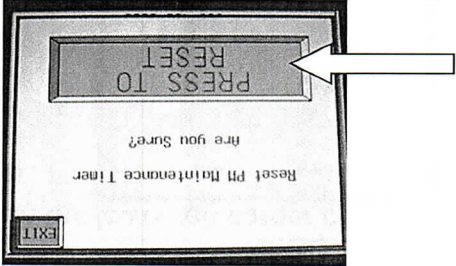


Step	Instruction
20.	Press Enter.
	
21.	Complete the checklist that was included with the kit.
22.	Return the checklist to <u>Zimmer Biomet</u> .

Ultra Duo Procedure:

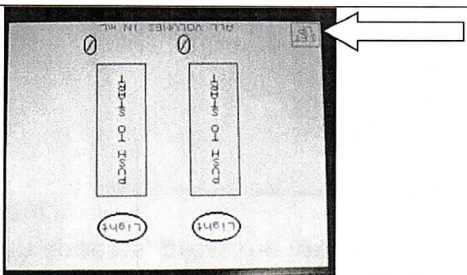
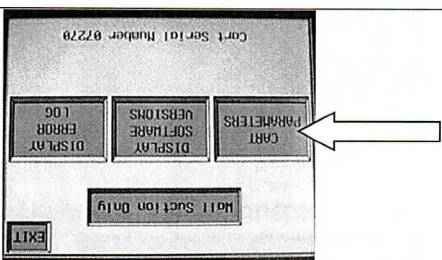
Step	Instruction
1.	To replace the two Teflon encapsulated O-rings use a 5/32" Allen wrench or equivalent and remove the four Allen head bolts that secure the face plate. 
2.	Remove the face plate and both receiver doors. Using a pick, small jeweler's screw driver, or equivalent, carefully remove the two encapsulated O-rings paying special attention not to damage the O-ring grooves. Install two new O-rings into the O-ring grooves located in the receiver housing, as indicated. 

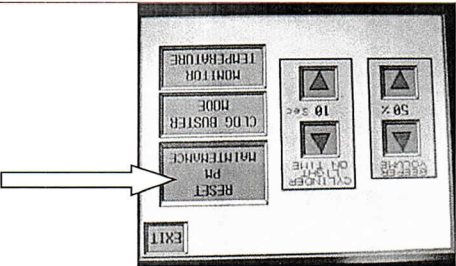
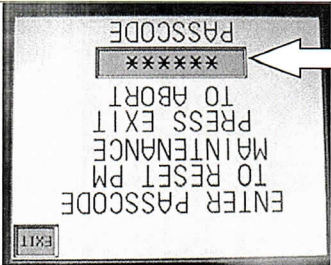
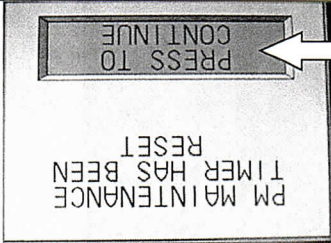
★ Personnel performing this procedure should wear appropriate personal protective equipment.

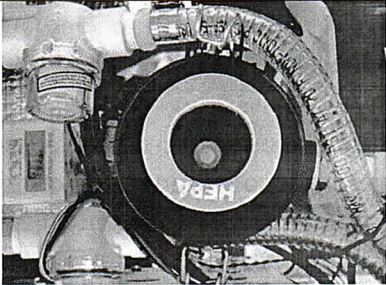
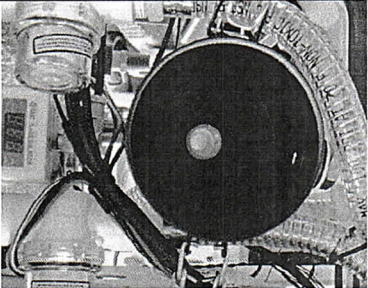
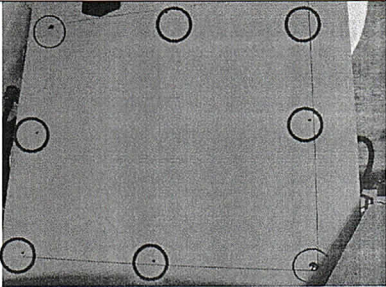
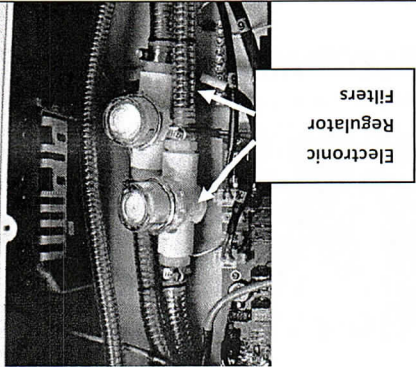
Step	Instruction
5.	 Press the PRESS TO RESET button to confirm.

6.	Exit out of the screens until the system restarts and ensure the PM timer has been reset.
7.	Complete the checklist that was included with the kit.
8.	Return the checklist to Zimmer Biomet.

Software 3.00, 3.10 3.60, 4.00 and 4.30

Step	Instruction
1.	 Press the SET UP button in the lower left corner of the screen.
2.	 Press the CART PARAMETERS button on the left side of the screen.

Step	Instruction	
3.	Press the RESET PM MAINTENANCE button in the upper right side of the screen.	
4.	Once the pass code screen appears, press the lower grey bar to bring up a key pad and input 147 followed by Enter.	
5.	Press the PRESS TO CONTINUE button to confirm.	
6.	Exit out to the main screen and then cycle the power to the unit to ensure the PM timer has been reset.	
7.	Complete the checklist that was included with the kit.	
8.	Return the checklist to Zimmer Biomet.	

Step	Instruction	
8.	Remove the cover to gain access to the filter.	
9.	Remove the filter and dispose of per the facilities guidelines.	
10.	Install new HEPA Filter and install the cover, and clamp back into position with the two clips. Ensure the cover is straight and centered before tightening the clips.	
11.	Disassemble the back panel by removing the 8-32 screws.	
12.	Remove the caps on the filter on each of the electronic regulators. Remove the filters and dispose of the filter per the facilities requirements.	

Step	Instruction
13.	Install the filters from the PM kit and then place the covers back on over the filters.
14.	Ensure the gasket O-ring is in the correct position when reinstalling the covers.
15.	Plug the unit into the wall and ensure that the unit has suction; unit should get to at least 575 mmHg with the black adapter to the lid plugged. Unplug the unit and hook it up to the Evac. Ensure that the unit processes successfully and that there are no leaks present.
16.	Plug the unit back into the wall and power up the unit. Reset the PM Timer per the correct instruction in the back of this packet. The method will vary depending on the software version.
16.	Reattach the lower front and rear lower panels. Reinstall the left and right kick plates.

Resetting the PM Timer on Ultra Carts Procedure:

Software 1.40 and 1.70

Step	Instruction
1.	On the main clinical screen, press the lower left corner of the display screen to make the pass code screen appear (there is hidden button located on the lower left hand corner of the display).
2.	Once the pass code screen appears, press the lower grey bar to bring up a key pad and input 15937 followed by Enter.

